DEUTSCH

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ENGLISH

Chemical composition: ZrO₂, HfO₂, Y₂O₃, Al₂O₃ & other oxides

Coefficient of thermal expansion (25-500°C):

Zirkon BioStar (colour) / Zirkon BioStar Ultra (colour, ML) –10,5 * 10⁻⁶ K⁻¹
Zirkon BioStar HT Smile (colour, ML) –10,0 * 10⁻⁶ K⁻¹

1. Indications

Zirkon BioStar (colour) / Zirkon BioStar Ultra (ML) (Ceramic: Type II, Class 5 as per ISO 6872:2019)

Zirkon BioStar (colour) / Zirkon BioStar Ultra (colour, ML) are dental mill blanks made from zirconium dioxide for the manufacture of dental prostheses. Zirkon BioStar (colour) / Zirkon BioStar Ultra (colour, ML) can be used for all dental constructions incl. bridges z 4 units with up to two connected pontics in the front and side tooth area. The indication for bridges in Canada is limited to six units with a maximum number of two pontics.

Zirkon BioStar HT Smile (colour, ML) (Ceramic: Type II, Class 4 as per ISO 6872:2019)

Zirkon BioStar HT Smile (colour, ML) are dental mill blanks made from zirconium dioxide for the manufacture of dental prostheses. Zirkon BioStar HT Smile (colour, ML) can be used for all dental constructions, bridges with up to 3 units (anterior and posterior region), inlays, onlays and veneers for dental application.

2. Contra-indications

Parafunctions, inadequate space, unsuitable preparation, incompatibility with components, inadequate oral hygiene.

3. Safety instructions

Do not inhale any milling dust during processing. Wear gloves, protective glasses and a face mask. (Follow safety data sheet)

4. Handling, storage and disposal

This medical product should only be processed by trained personnel. Do not use any materials which have been damaged. Only store the blanks in the original packaging in a cool, dry environment. Dispose of any waste in accordance with national and local regulations.

5. Construction and milling (CAD/CAM)

Construction and sizing should be carried out with consideration given to subsequent requirements of the technical dental work in the patient's mouth. Wall thickness and connection cross-section should be adequately stable. We recommend use of a chamfer or stepped approach. Any machine processing should only be carried out using the milling system intended for use with the product. The machine manufacturer's instructions must be followed.

6. Colouring, sintering and re-working

The framework can be coloured before final sintering with DD Shade Concept® (follow separate usage instructions). We recommend a temperature of 1450 °C for sintering (follow sintering instructions). Avoid any subsequent re-working.

7. Ceramic veneers

Veneer with a material suitable for the coefficient of thermal expansion of the zirconium dioxide. Follow the ceramic veneer's usage instructions.

8. Fixing

Conventional cementing with zinc oxide phosphate cement or glassionomer ce-

ment. Adhesive fixing for small retention surfaces. A temporary mounting is not recommended!

For more detailed processing information, please see the current version of the instructions for use in each case and the safety data sheet in the download centre at: www.siladent.com.

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Note: Cantilever pontics must not be longer than 2/3 of the supporting crown width.

** The indication for bridges in Canada is limited to six units with a maximum number of two pontics.

- Please pay attention to our separate sintering instruction!**
Sintering cycle for normal furnace filling without cover:
- ↑ Heating up to 900°C (1652°F) with 8°C/min (46°F/min),
→ Dwell at 900°C (1652°F) for 30 minutes,
 - ↑ Heating up to final temp. 1450°C (2642°F) with 3 °C/min (3 °F/min),
→ Dwell at final sintering temperature 1450°C (2642°F) for 120 min,
 - ↓ Cooling to 200°C (392°F) with 10°C/min (50°F/min).

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